

INSTRUCTIONS FOR USE



Vizulize Vitamin A & E Eye Drops have been specially developed to provide effective and long lasting relief for dry, tired and irritated eyes while also supporting eye health and vision.

The specially developed formula includes:

Sodium Hyaluronate to moisturise, hydrate and soothe dry eyes and the associated feelings of irritation, grittiness and discomfort.

Compatible with contact lenses.

MODEL DETAILS

Moisturising and Comfort drops **Formulation F**

INGREDIENTS

0.1% Carmellose, 0.05% sodium hyaluronate, 1% polysorbate 80, 0.015% retinyl palmitate (Vitamin A) and 0.015% tocopherol acetate (Vitamin E)

DIRECTIONS FOR USE

- Ensure your hands are clean and dry before use
- Shake bottle before use
- Check the bottle seal is not broken or missing before use.
- To open the bottle: Twist the cap anti-clockwise to break the seal.
- Tilt your head backwards and gently squeeze 1 to 2 drops into the eye
- Blink several times to cleanse the eye
- Repeat for the other eye as necessary.
- Replace cap after use.
- May be used prior to and following contact lens wear.
- Use as often as required / safe for everyday use
- Compatible with contact lenses

WARNINGS:

The product is sterile until first opened

- Do not use if tamper-evident seal is missing or broken.
- Do not swallow the solution
- If eye irritation occurs, discontinue use and consult an eye care practitioner. If condition worsens, consult a specialist.
- If you have been diagnosed with a medical eye condition or are using other eye medication, consult a healthcare professional before use.
- Do not use if allergic to any of the ingredients.
- May cause temporary blurring of vision, if affected do not drive or perform skilled tasks until vision is clear.
- For use in the eyes only
- This product should not be used to soak or disinfect contact lenses
- To avoid contamination, do not touch the dropper tip.
- Do not use after the expiry date

INSTRUCTIONS FOR USE



In the event of a serious incident occurring during the use of this medical device or as a result of its use, please report the incident to the manufacturer and the Competent Authority of the Country in which you are located

For adults - May be used by children younger than 16 years under supervision

KEEP OUT OF SIGHT AND REACH OF CHILDREN.

AFTER OPENING

- Use within 90 days of first opening / Once opened, discard after 90 days

STORAGE

Store between 5 °C and 30 °C

Avoid direct sunlight.

Do not use if package is open or damaged



East Midlands Pharma Ltd
Unit 1 Rothley Lodge, Loughborough Road
Rothley, Leicestershire, LE7 7NL, UK



Advena Ltd, Tower Business Centre,
2nd Flr, Tower Street, Swatar, BKR 4013
Malta



STERILE A



2797

For any additional questions on the use or safety of this product, please contact us on
technical@empharma.co.uk

Revision History

Rv	Date	Description
01	08 May 2026	New electronic Instructions for Use (Visualize brand)
02	15 Aug 2026	Added additional warning on For adults - May be used by children younger than 16 years under supervision to align with the technical and clinical data