

VIZhyal 0.4% Product Information

VIZhyal 0.4% is a sterile, ophthalmic soothing and moisturising solution, with a physiological pH, based upon cross-linked sodium hyaluronate. This gives more prolonged lubrication than with standard sodium hyaluronate and has a re-epithelizing effect for greater protection of the corneal surface.

VIZhyal 0.4% does not contain preservative. The absence of preservatives allows the use of the solution by contact lens wearers without the need of removing them.

Composition

Sodium hyaluronate cross-polymer aqueous solution (of which sodium hyaluronate cross-polymer 0,4%), Trometamol, Hydrochloric acid 2N, Sodium chloride, water for injection.

Indications

VIZhyal 0.4% relieves redness, dryness, fatigue and discomfort to the eyes, and ocular mucosa caused by:

- dry eye syndrome
- environmental factors such as wind, smoke, pollution, prolonged exposure to sunlight, dried salt, sea water or pool water
- dryness of the eyes due to excessive fatigue caused by working on the computer for lengthy periods, study or driving;
- inflammations, eye surgery or eyelid aesthetic surgery and chronic blepharitis.

VIZhyal 0.4% hydrates and lubricates the cornea and the conjunctiva in case of dry eye due to the use of contact lenses.

VIZhyal 0.4% is indicated for: for reinstatement and improvement of tear film stability in dry eye syndrome after refractive surgery; lacrimal abnormalities secondary to inflammations, chronic blepharitis; lacrimal abnormalities secondary to eye surgery of various nature or eyelid aesthetic surgery.

Common Side Effects

Due to its viscosity, the product can cause a short-term blurring of vision after use. The patient should be advised not to drive or use machines until their sight is clear.

Minor burning or stinging may temporarily occur as a result of ocular irritation.

Dosage or instructions for use

Instil 2 drops in each eye.

It is suitable for everyday use, daily and night-time.

Suitable for frequent use. Can be used 2 or 3 times, or more often if required.

The gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the properties of the closure system act to preserve the sterility, and the chemical and physical characteristics of preservative-free solutions.

Warnings

The product is for single patient use.

The product is only intended for ophthalmic use.

Care should be taken during application, not to touch the eye and any other surface with the dropper of the container.

Contraindicated in patients with a history of hypersensitivity to any of the components.

Keep away from children.

Legal classification

Medical Device Class 2b

Presentation

10ml drops

Do not store above 30°C.

After the first opening the content must be used within 3 months.

Date of last revision

Feb 19

Adverse Events

All adverse events should be reported. Reporting forms and information can be found at www.mhra.gov.uk/yellowcard or search for MHRA Yellowcard in the Google Play or Apple App store. Adverse events should also be reported to EM Pharma on 01664 820347 or via customercare@empharma.co.uk